


HYSTERA Electronic system 1.0

SETUP MANUAL


Once you download your copy, please:

1. Extract the downloaded file directly into your “C:” drive using winrar:


First: choose the folder “Hystera Electronic System version 1.0” and click “Extract to”:

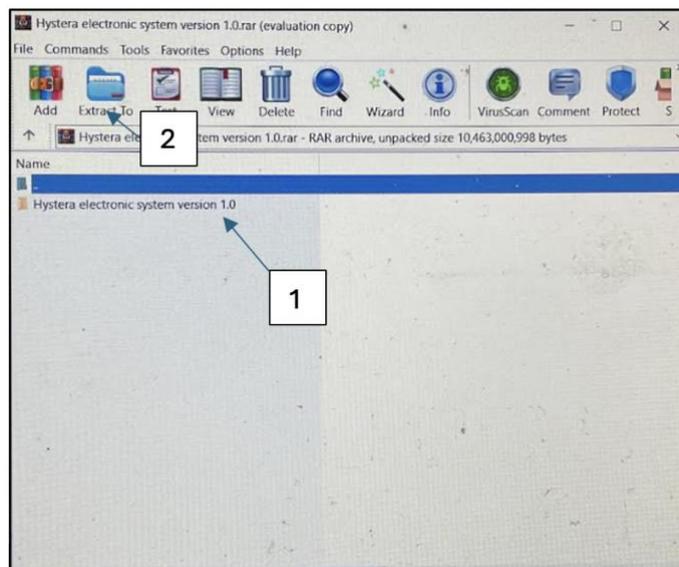

Second: write “c:\” in the destination path and then click “Ok”

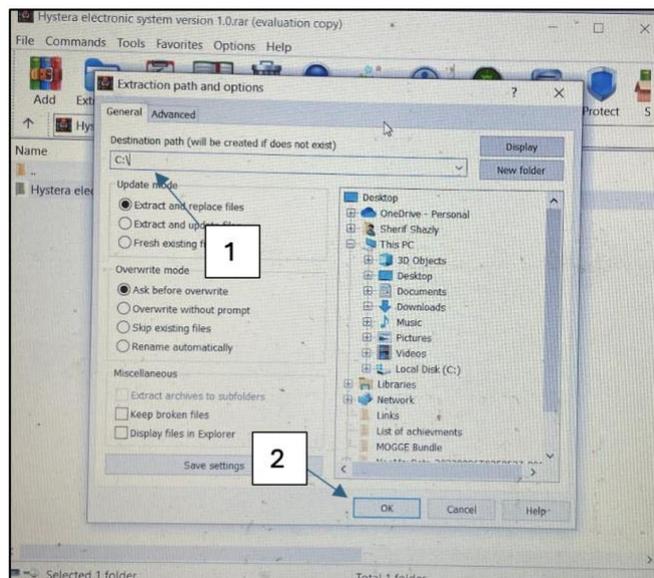

2. You will now find the folder named “C:\hystera Electronic System version 1.0” on your computer. Be sure it looks like that and is not 2 folders with the same name inside each other
3. Now, please open “Console - offline” if you are using offline version or “Console” if you are using online version (files with the black icon):

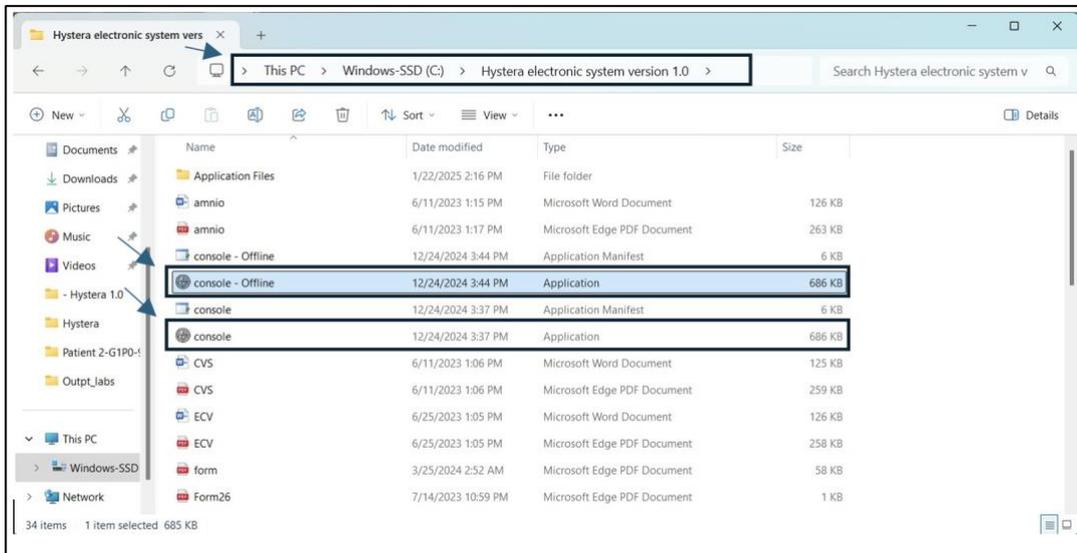

To ensure the application works correctly, please:

- 1- Always open “the console” when you start using the application and allow it to load. You can then click the first button on the left to launch “Hystera main application”
- 2- After you finish, close the hystera main application and always close “the console” at the end of your work to ensure correct saving of your work
- 3- Use English language only when you put the name and the clinic number of your patients to be saved correctly.
- 4- Do not share your copy with other people since they could disturb file saving process and mix data.